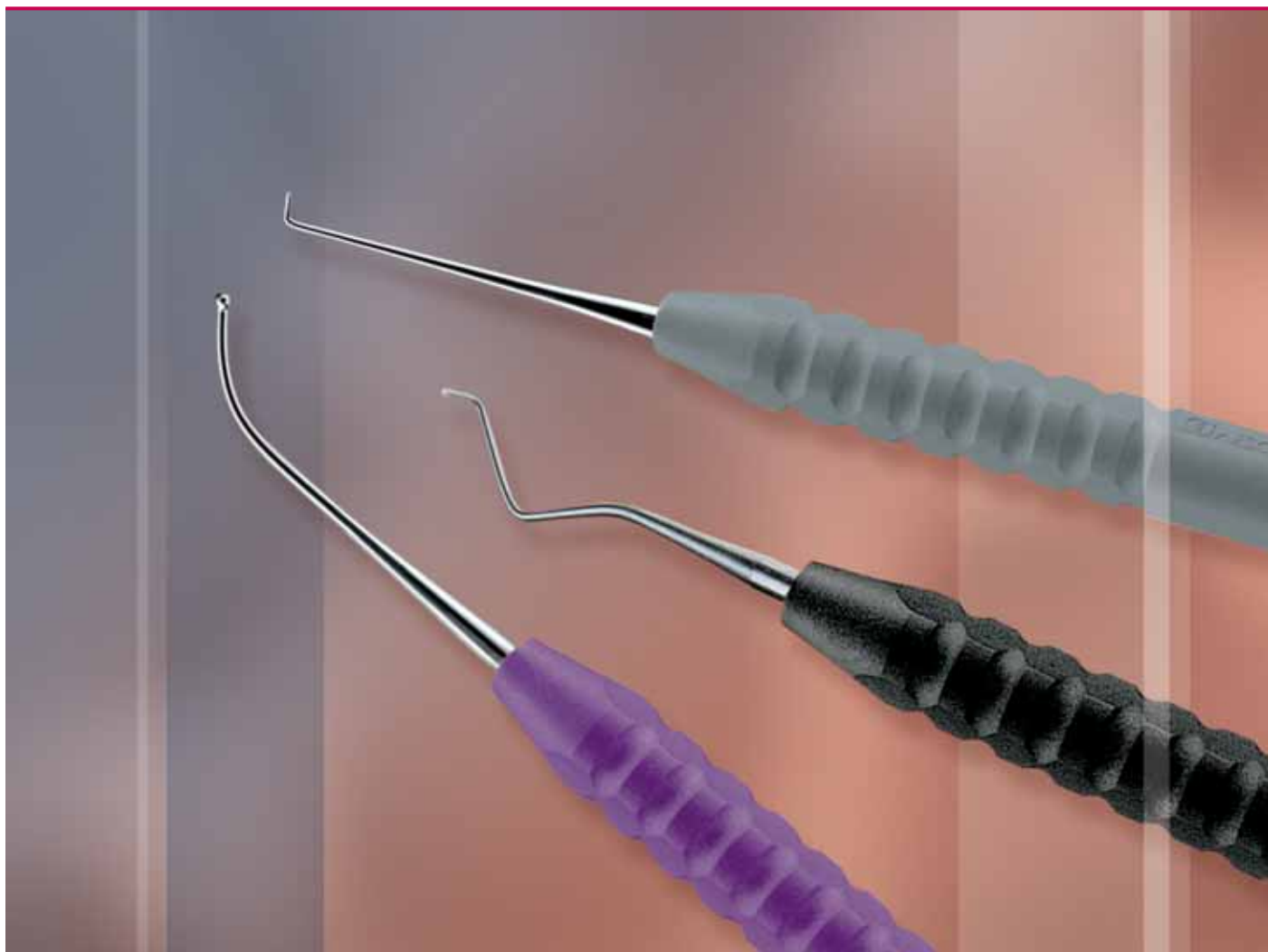
Aesculap Dental

Endodoncja

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Dentistry is currently facing drastic changes. The current state of medical and technological knowledge means that operations can be performed with ever greater precision and using less and less invasive techniques. Along with the availability of increasingly sophisticated diagnostic tools, the decisive factor underlying this progress is the parallel ongoing development of the entire range of dental instruments.

With its new and innovative products, Aesculap is playing a decisive part in this development.

Aesculap's products are developed strictly by the close contact to our customers.

This applies as much to new and innovative areas such as microsurgical instruments for implantology, periodontics and endodontics, as to standard dental instruments in general. Throughout the world, Aesculap products have proved themselves in day-to-day practice, not least from the point of view of quality and cost. With limited investment budgets available,

optimized hospital procedures and resulting efficiency gains offer a means of making better use of existing resources. Aesculap's range of services can help to achieve such improvements.

The following service components enable us to meet the ever increasing demands of the future:

- our many years of experience and competence in the field of planning and advice,
- the world renowned excellent quality of our products,
- our high level of product availability,
- individual and practice-oriented service concepts to maintain the value of your dental instruments,
- continuous new opportunities for ongoing and further training in the form of the courses offered by the Aesculap Academy.

Because of the excellent therapeutic results achieved, micro-endodontics is gaining in importance.

Aesculap did pioneering work in this field years ago, offering a comprehensive range of instruments specially designed for the new demands of microsurgery and micro-endodontics. Today it is barely possible to imagine working under the microscope without new developments such as our outstanding micro mirrors. The extremely flexible combined micro mirror and scalpel blade holder is also being used successfully in other dental disciplines.

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Elevation of the flap starting from the relieving incision with the disk-shaped elevator (DB407R).

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Retraction of the flap using the
 retractors (D0371R resp.
 D0372R)

The apical lesion is visible in the
 center of the picture.

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The apical lesion removed using the following: disk-shaped curettes (DB406R resp. DB408R, the spoon-shaped curette (DB409R) or the angular scaler (DB410R).

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Diamond Powder Coated Forceps

- non-traumatic and yet firm gripping of sutures and tissues

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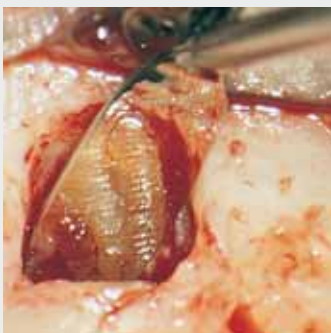
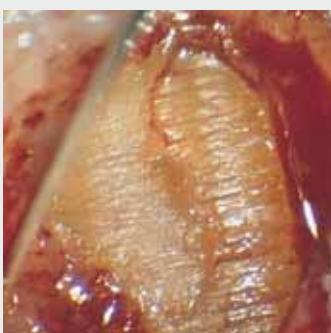
Use of the oval micro mirror (DA045R resp. DA046R).
Fracture line barely detectable in low magnification (12x).

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Large magnification (25x) of the above illustration.
The fracture line is visible in the center of the illustration.

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Use of the round micro mirror (DA043R resp. DA044R). View of prepared retrograde cavity with the post in the root canal.

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Micro mirror, 2 mm wide, 45° bevel, interchangeable

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Micro mirror, 3 mm wide, 45° bevel, interchangeable

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Micro mirror, round diam. 3.5 mm 35° bevel, interchangeable

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Micro mirror, round diam. 4.7 mm 35° bevel, interchangeable

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Micro mirror, round diam. 3.5 mm
 30° bevel, interchangeable

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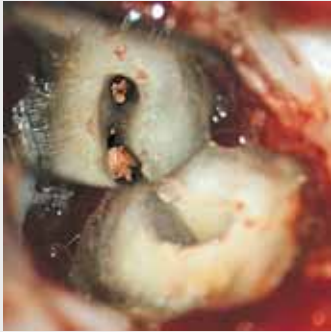
Oval retrograde cavity, seen after preparation of the isthmus between the root canals. The oval shape plugger (DE442R) is used to condense the retrograde filling material.

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Application of the retrograde filling material (Super EBA) into the cavity with the micro spatula (DE440R resp. DE441R).

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Ball burnisher/spatula, Micro spatula, angulated

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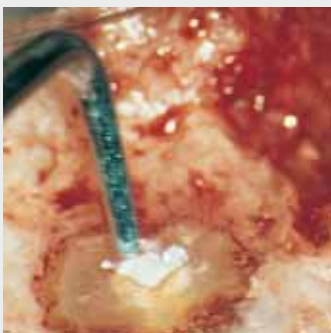
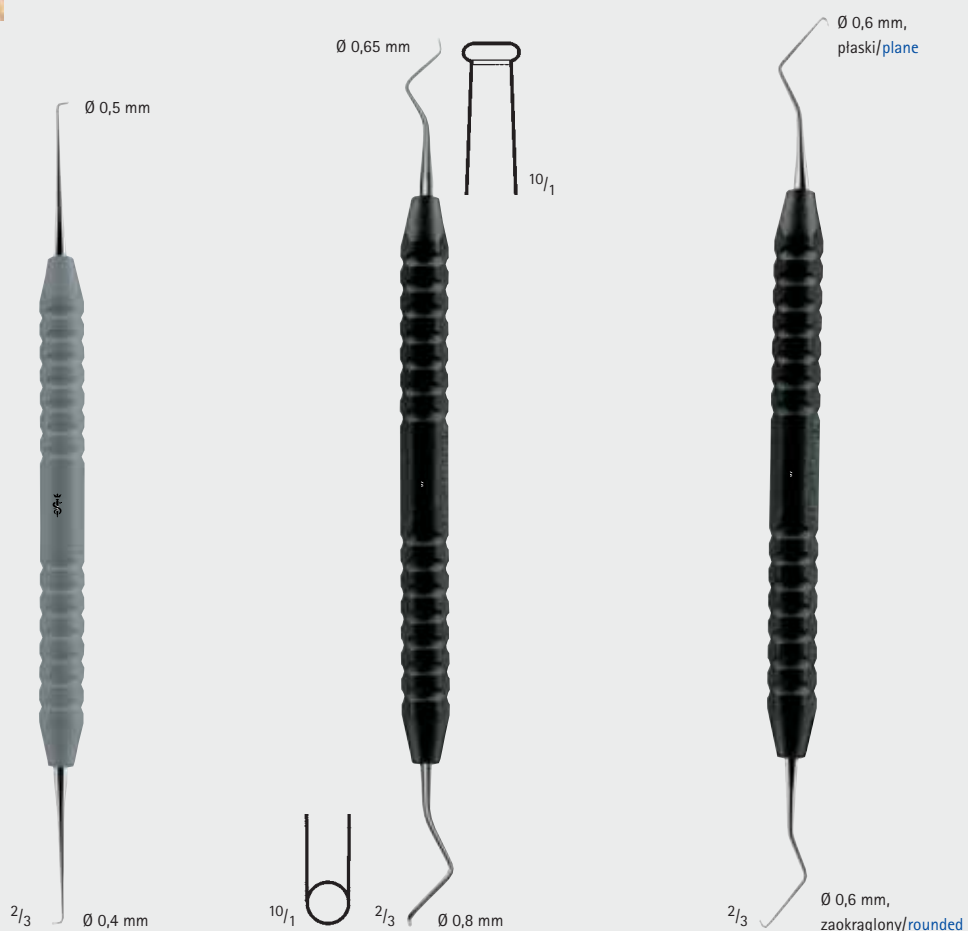
Ball burnisher/spatula, Mikro spatula, straight

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Condensation of the filling material with the round plugger (DE444R resp. DE445R or DE443R) (in round canals).

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Plugger, round with angulated shaft and tip, one tip is undercut

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Plugger, round with angulated shaft and tip

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Burnishing of the filling material. With the ball burnisher (DE440R resp. DE441R).

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Removal of excess filling material with the micro carver (DB411R).

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Ball burnisher/spatula, angulated
 175 mm, 7"

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New micro instrument handles

As well as its optically appealing design, the new surface we have created for the handles of our successful micro instruments offers principally hygienic advantages. The substantially improved grip ensures a more fatigue-free working

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Scissors

Screw connection providing:

- smooth and reliable function
- longer life time by the possibility of regrinding

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Micro scissors (spring type), curved, sharp/sharp

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Special hint

Since suture material is rather tough, the cutting capacity of scissors is quickly reduced which is noticeable as soon as tissues are cut afterwards.

Our solution

To cut sutures with special ligature scissors. The ligature scissors FD252R have additionally rounded edges to avoid sutures to get jammed.

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Micro ligature scissors, curved, sharp/sharp, one blade serrated, for suture material 6/0 – 8/0

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Diamond Powder Coated Needle Holders

- optimized gripping of needles and sutures
- long life time, since the degree of the temper of the diamond powder coating is nearly identical to the temper of the needles
- for suture material 6/0 – 9/0

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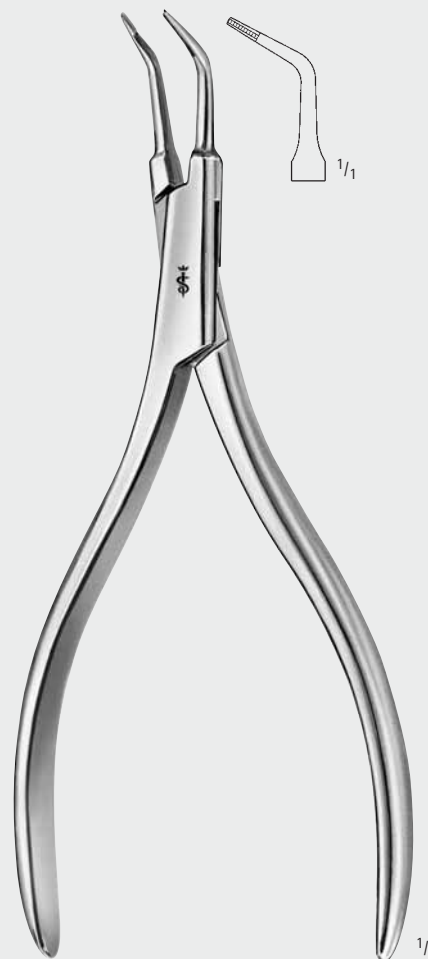
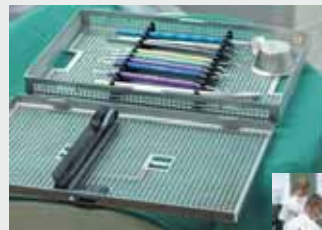
The sterile goods cycle in the dental practice with the Aesculap Sterile Container System

All hygienic measures in the dental practice are primarily for keeping healthy the patients, the dentist and the assistants.

To ensure the best possible safety it is particularly important to observe the principles of hygiene when reprocessing instruments.

To keep practice work safe, efficient and economical, clear procedures must be specified for practice hygiene.

The Aesculap Sterile Container System has proven itself as an excellent aid for organizing the instrument reprocessing procedure.

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All rights reserved. Technical alterations are possible. This leaflet may be used for no other purposes than offering, buying and selling of our products. No part may be copied or reproduced in any form. In the case of misuse we retain the rights to recall our catalogues and pricelists and to take legal actions.

B | BRAUN
SHARING EXPERTISE

Aesculap Chifa Sp. z o.o.

ul. Tysiąclecia 14
64-300 Nowy Tomyśl, Poland
tel. +48 61 44 20 100
fax +48 61 44 23 936
info.acp@bbraun.com
www.chifa.com.pl